

# SCOPE BUDDY®

Endoscope Flushing Aid | **Manual Clean\***



\*SCOPE BUDDY is only intended to be used as a Flushing Aid during Manual Cleaning step of Endoscope Reprocessing. It is not intended to manually clean endoscopes.



## THE COMPLETE CIRCLE OF PROTECTION

As the global vanguard in infection prevention, only Cantel Medical delivers the Complete Circle of Protection, a full-value, proactive partnership dedicated to helping you remove risk, streamline operational efficiencies and optimize your success.

## Consistent, Hands-free Flushing of Endoscopes

### Reduces the Risk of Repetitive Motion Injuries

- Replaces repetitive and labor-intensive syringing of channels
- Simple, hands-free operation reduces muscle fatigue
- Provides constant fluid delivery through all endoscope channels

### Provides Consistent Flushing

- Continuous, timed fluid delivery ensures consistent, repeatable endoscope flushing
- Audible and visual alerts upon completion
- Compatible with all major endoscope brands
- Works with your existing sink; compact size for countertop or wall mounting
- Meets or exceeds endoscope manufacturers requirements for channel flushing volumes

### Shortens Flushing Time

- Thoroughly flushes most endoscopes in 35 seconds
- Flushes multiple endoscope channels simultaneously
- Quickly and quietly delivers fluids through endoscope channels

### INTERCEPT® Detergent Provides Superior Endoscope Cleaning

- Fast, one-minute contact time
- Low foaming, neutral pH formulation safe for endoscope use
- Cost-effective concentrated formula reduces per-use costs



## MANUAL CLEAN



Manual cleaning is essential in removing bioburden and preparing the endoscope for high-level disinfection. For optimal results, Cantel's manual cleaning products and detergents are designed to be a critical first step to efficiently and effectively eliminating bacteria on endoscopes prior to reprocessing.

# SCOPE BUDDY®

Endoscope Flushing Aid | **Manual Clean\***



Visual display of endoscope flushing time

User-selectable time setting

Audible and visual alerts upon completion

One-touch start/stop operation

Simple and quick tubing connections

\*SCOPE BUDDY is only intended to be used as a Flushing Aid during Manual Cleaning step of Endoscope Reprocessing. It is not intended to manually clean endoscopes.

## Complete Endoscope Connection Kit



Wall Bracket

Power Supply

### SCOPE BUDDY® Flushing Aid

#### Unit Dimensions

9.5 H x 6 W x 8 D (inches)  
 (24.1 H x 15.2 W x 20.3 D (cm))

#### Unit Weight

10 lbs (4.5 kg)

#### Electrical Requirements

110 – 230 V, 50/60 Hz

#### Electrical Safety Certifications

UL Standard 61010-1  
 CANADA/USA C22.2 No. 61010-1  
 ETL and cETL listed

#### Fluid Delivery Rate

1 liter/minute

#### Flush Time

0 – 99 minutes

#### Timer Control

Digital display with touchpad controls

#### System Pressure

Built-in safety bypass

#### Cycle Completion Indicators


Audible and visual alerts

#### Unit Placement

Counter-top or wall-mounting  
 (bracket included)

#### Warranty

One year

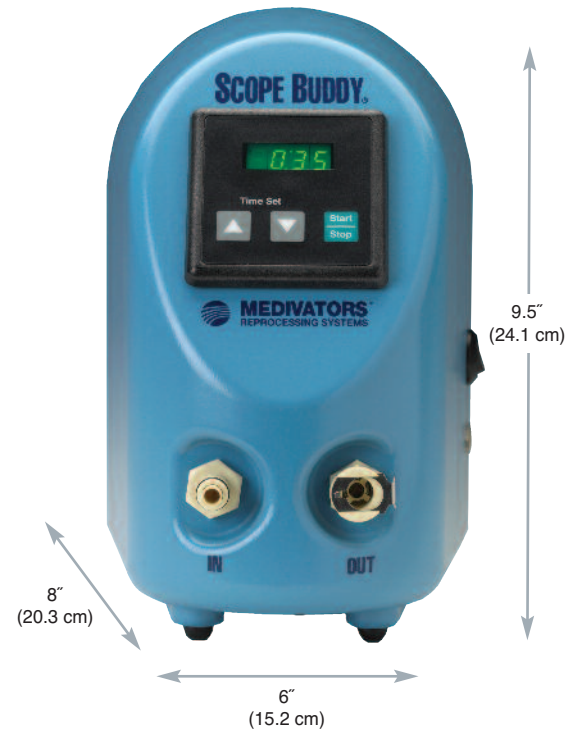

### SCOPE BUDDY Flushing Aid Ordering Information

Model Number	Description
EFA-US-G	North America
EFA-IN-G	International

### Accessory Ordering Information

Part Number	Description
78399-669	Complete Endoscope Connection Kit
75227-679	Graduated Cylinder

### Detergent Ordering Information

Part Number	Description
ML02-0106	INTERCEPT® Detergent (4 one-gallon bottles/case)

SCOPE BUDDY® and INTERCEPT® are registered trademarks of Medivators Inc.



(949) 573-1534  
 info@vendihealth.com  
 www.vendihealth.com